

PREVAIL® HOSPICE PROGRAM

QUIZ Answer sheet

1) Which of the following determine the appropriate Prevail® product to be used?

- a. Mobility
- b. Type of Incontinence
- c. Level & Frequency of urinary incontinence
- d. A, B, C—Determine the correct level of product absorbency based on patients using the assessment process.**

2) Which product would be best for a female patient who has good mobility and light urinary incontinence?

- a. Adult Brief
- b. Protective Underwear
- c. Bladder Control Pad—designed for lighter urinary absorbency, patients who experience “dribble” to small leakage, often related to stress or urge incontinence.**
- d. Underpad (Chux)

3) Bladder Control Pads & Male Guards should be used as an insert inside protective underwear or briefs.

- a. True
- b. False—Bladder control pads & male guards are designed to be used in the patient’s own undergarment. They should NEVER be used as liner or insert, this is considered double padding. Double padding will lead to more leakage, increase risk of infection & moisture build up.**

4) Protective Underwear are for patients who experience moderate urinary leakage and bowel incontinence.

- a. True
- b. False—Protective Underwear are designed for light to moderate urinary leakage, they are not designed for bowel containment.**

5) PER-FIT 360™ Expandex™ Wings are a 2 tab system.

- a. True—they are a 2 tab system using soft, flexible stretch panels, creating a snugger & secure fit.**
- b. False

Continue to Back

PREVAIL® HOSPICE PROGRAM

QUIZ Answer sheet

6) Which of the following allow the PER-FIT 360^{OTM} Next Generation Brief to create a favorable microclimate?

- a. 360° Breathable Zones
- b. Doublesorb™ Layers
- c. QUICK WICK® with MaxSoft™
- d. All of the above—Favorable Microclimate Care™ is a combination of all 3 features, effectively reducing moisture/humidity from the skin surface while helping to maintain normal skin temperatures.**

7) Prevail® Super Absorbent Underpads should be NOT be used for which of the following:

- a. Protecting bed during wound change
- b. In place of an incontinent product and leaving patient open to air--Prevail® Underpads are NOT designed to be used in place of an incontinent product.**
- c. A & B

8) Prevail® Adult Washcloths are flushable.

- a. True
- b. False-Prevail® Adult washcloths are designed for peri-care after incontinent episodes. They should NOT be used for cleaning surfaces, tables, doors, hands, and faces or flushed in the toilet.**

9) When applying the PER-FIT 360^{OTM} briefs and using the correct size and application technique the tabs will be placed:

- a. On the abdominal area
- b. On the hips-If tabs are placed on the abdominal area the product is too large. The product will not fit correctly; patient will experience leakage and possible skin irritation from improper size and fit. The patient should be resized using the Prevail® sizing chart.**

10) Reduce utilization by doing the following:

- a. Select appropriate Prevail® product based on individual needs
- b. Determine the correct size using the Prevail® sizing charts
- c. Use correct application technique when applying Prevail® products
- d. All of the above- Using the correct level of absorbent product, size & application technique ensures product performance and patient comfort; leading to improved utilization.**